

Luise C.¹, Senic-Matuglia F.¹, De Marni M.L.¹, Moretti V.¹, Stepien G.E.¹, Gole L.¹, Chen S.¹, Picocci S.¹, Mazzarella L.² and Bascetta L.¹

¹Tethis S.p.A., Via Olgiati 5, 20143 Milan, Italy; ²Istituto Europeo di Oncologia, Via Adamello 16, 20139 Milan, Italy

Introduction

CTCs and cfDNA capture tumor heterogeneity and evolution beyond single-site tissue biopsies¹. Our study evaluates a novel liquid biopsy workflow in Metastatic Breast Cancer (MBC) by comparing their biomarker profiles with matched metastatic tissue.

See.d® is a liquid handler that automates pre-analytical processing of blood samples² collected in EDTA, providing:

- **nucleated cell fraction**, white blood cells (WBCs) and rare CTCs, seeded onto Tethis' **SmartBioSurface® (SBS) slides**³⁻⁴⁻⁵
- **plasma** collected for molecular analysis

Matched **plasma samples** prepared with See.d® and **Cell-Free DNA BCT®** tubes were also analyzed using a commercial NGS panel to compare cfDNA mutations (Thermo Fisher).



Methods

Patients	10 MBC patients (40% ER+, 20% ER+/HER2+, 40% TNBC) ⁶
Blood Processing	- Streck Cell-Free DNA BCT®: plasma collected using standard protocol - See.d® for plasma in EDTA and SBS slides (cytology slides)
CTC Analysis ⁷⁻⁸	- IF staining (PanCK/EpCAM + ER + CD45/CD66b) - H&E staining for morphological evaluation - ERBB2 FISH (for ER+/HER2+ and TNBC histotypes)
DNA Profiling	- Plasma: OncoPrint Dx Express panel (Thermo Fisher) - Tissue: OCA Plus panel (Thermo Fisher)
Definitions	- Multimodal tumor informativity : ≥1 CTC (identified by IF and confirmed by cytologist) - Concordance with Tissue Biopsy : Biomarker expression : ≥1 CTC with ER expression or ERBB2 amplification Mutation profiles identified by NGS : Full concordance: plasma shares all or additional mutation with tissue Partial concordance: plasma shares only some mutation with tissue

CTC



FIGURE 1: Liquid Biopsy workflow for Sequential Staining and Multimodal Imaging. SBS slides allow the use of multiple staining modalities: Immunofluorescence (IF), Hematoxylin and eosin (H&E), and Fluorescence In Situ Hybridization (FISH) on a single slide; after multimodal staining, a gallery of images is provided for analysis.

PLASMA

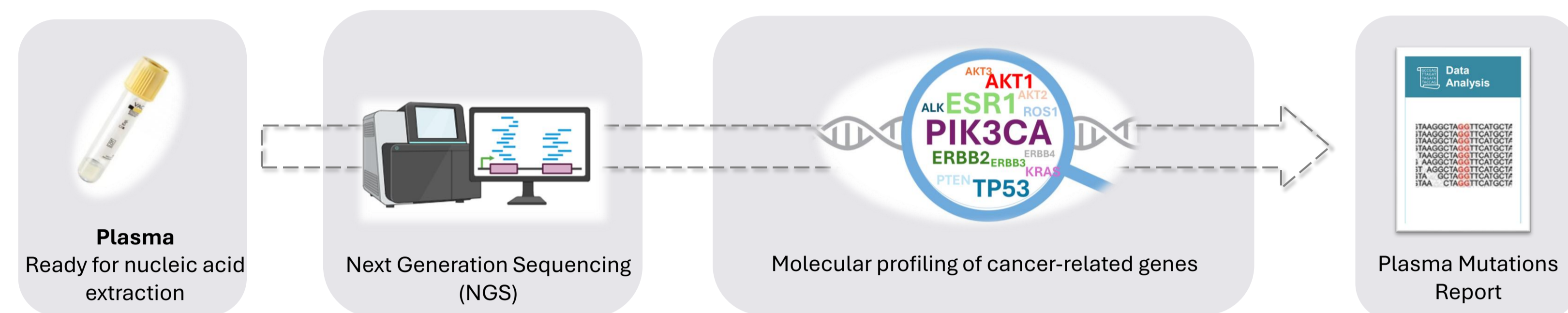


FIGURE 2: Plasma in EDTA collected by See.d®. Upon See.d® processing, plasma is stored at -80°C for cfDNA extraction, quality and NGS analysis with commercial panels

Results

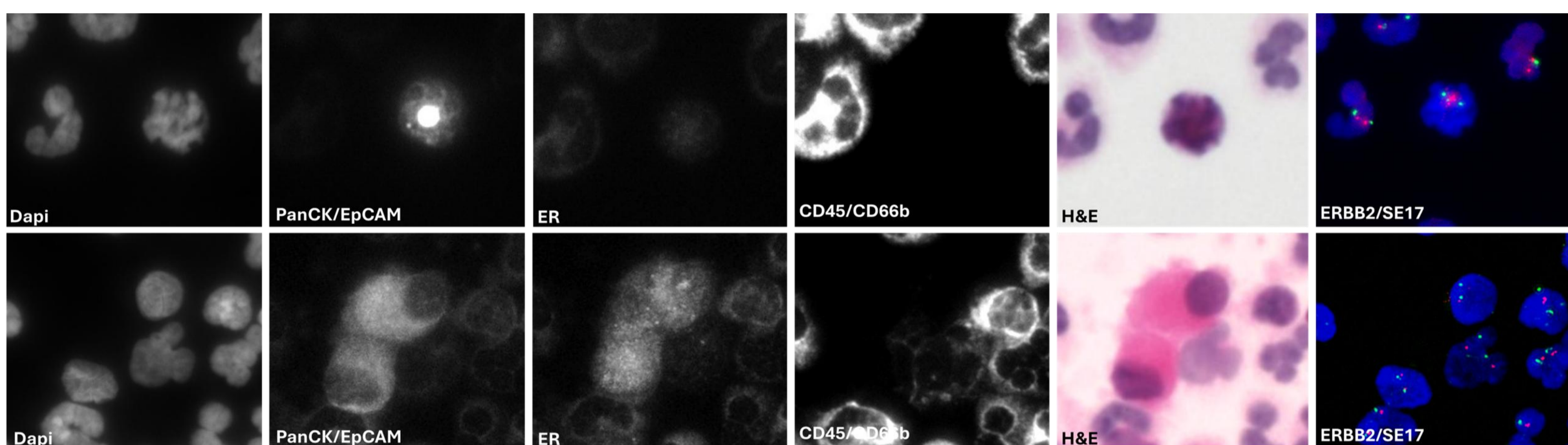


FIGURE 3: Example of multimodal staining in 2 different patients. Upper panel: patient PT02 with tissue histotype ER+/HER2+ shows no ER expression at single cell level, malignant chromatin features evidenced H&E stain and ERBB2 FISH amplification; Lower panel: patient PT10 with tissue histotype TNBC shows a cluster of two cells with ER expression, malignant morphology revealed by H&E staining and no ERBB2 FISH amplification.

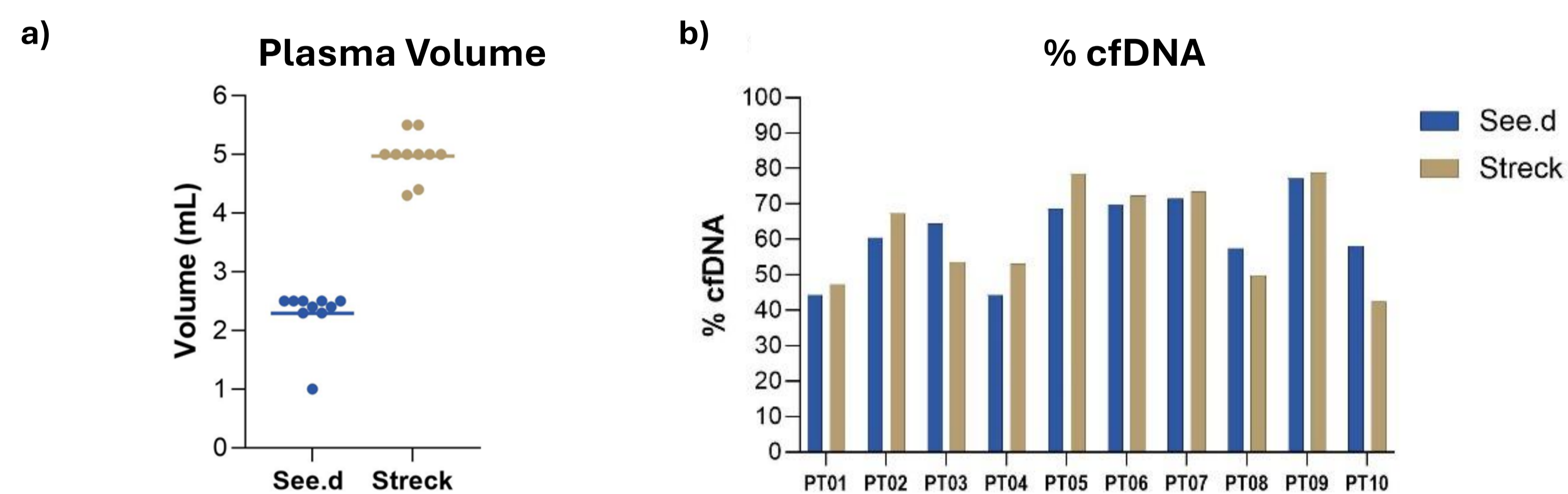


FIGURE 5: Comparison of plasma yield and cfDNA quality. a) Graph showing the **volume of plasma recovered** after See.d® and manual standard processing (Streck). b) Graph showing the **percentage of cfDNA** (range 50-300 bp) in samples processed on Tape Station run (size range 50-700 bp) by See.d® and with manual standard processing (Streck).

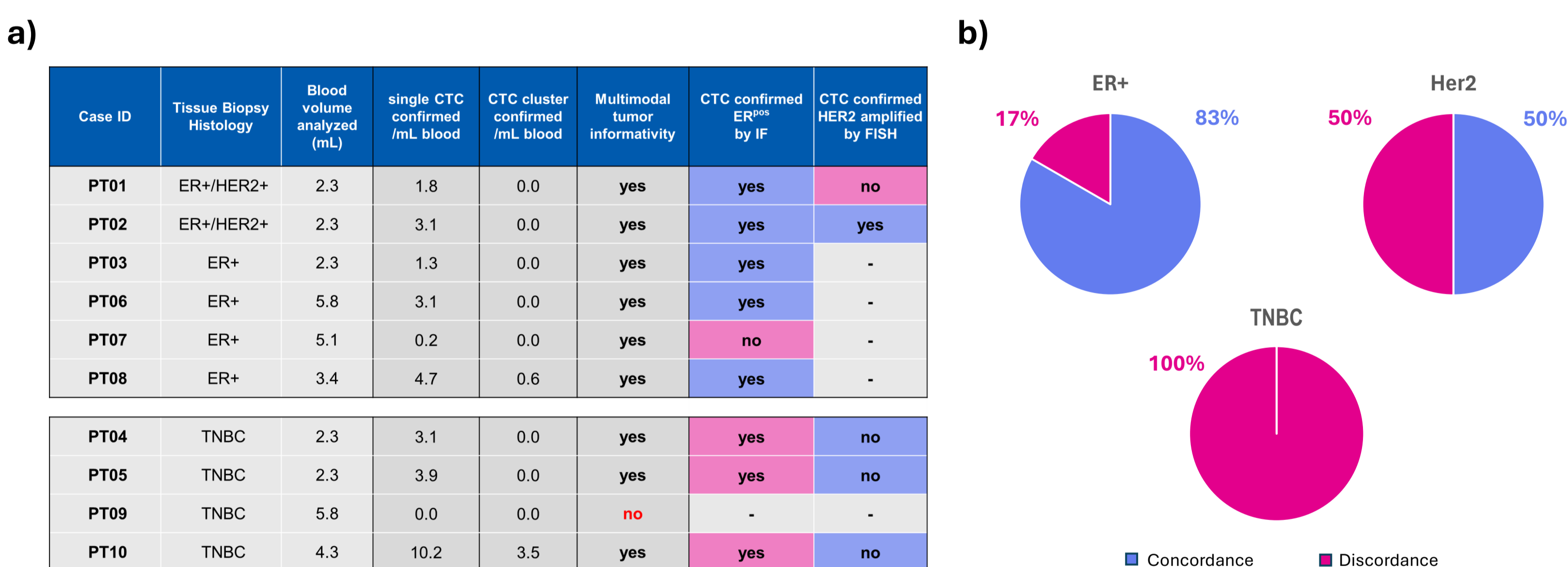


FIGURE 4: a) CTC enumeration and characterization: the table summarizes, for each enrolled patient, the tissue biopsy histology, blood volume analyzed, tumor-informativeness status and enumeration, and biomarker expression of cytologist-confirmed single cells and cluster CTCs; **b) Concordance of biomarker expression between tissue biopsy and CTCs.** Pie charts illustrate the proportion of matched and unmatched expression profiles.

CTC Analysis and Biomarkers Concordance with Tissue Biopsies

- **CTCs were identified in 90%** of cases using the multimodal analytical approach
- **Concordance** of CTCs biomarkers characterization with matched tissue was **83% for ER** and **50% for HER2**
- No concordance was observed in TNBC samples

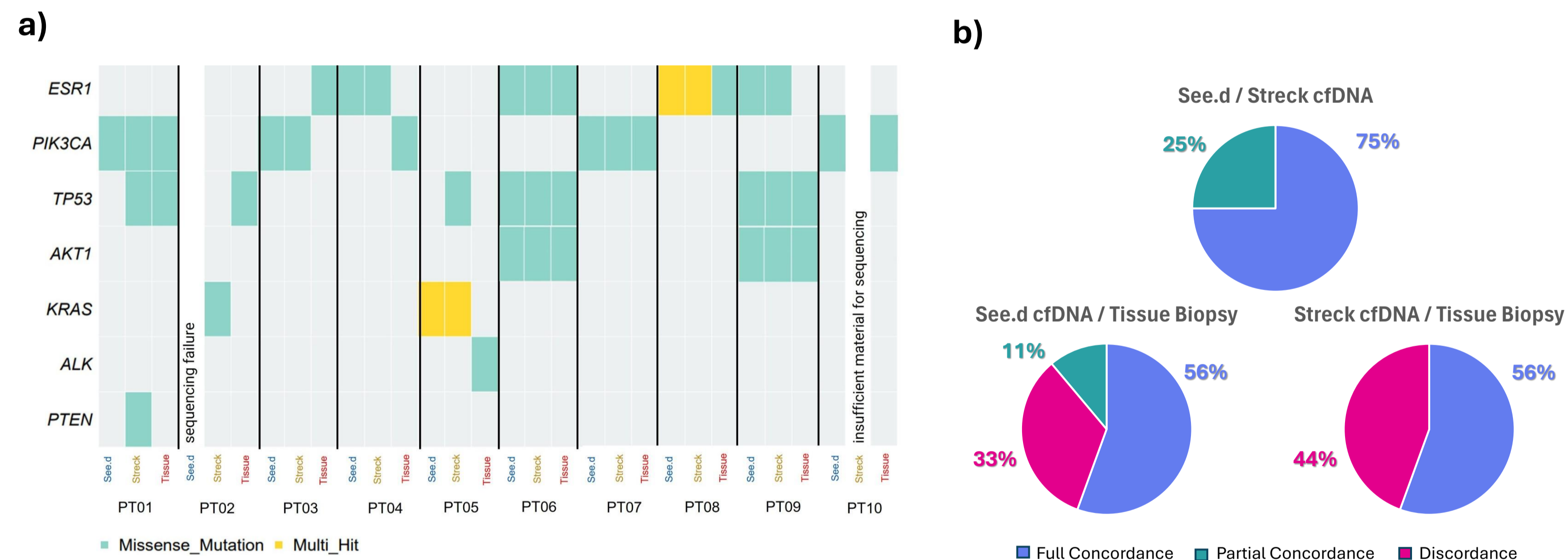


FIGURE 6: a) OncoPrint showing somatic variants detected by sequencing of cfDNA from plasma (processed with See.d® or using the standard manual protocol in Streck tubes) and gDNA from tissue biopsies. Rows represent genes; columns represent See.d®, Streck, and Tissue samples grouped by patient (PT01-10). Mutation types are color-coded: missense (cyan) and multi-hit (yellow). **b) Concordance of mutation profiles.** Pie charts illustrate the proportion of matched and unmatched genetic alterations between See.d® and Streck samples and See.d®, Streck, and tissue biopsy.

cfDNA Concordance between See.d® and Streck (8 evaluable cases)

- Full concordance: **75%**; partial concordance: **25%**

cfDNA vs. Tissue Biopsy Comparison (9 evaluable cases):

- See.d®: full concordance in **55.5%** (PT02 excluded for sequencing failure)
- Streck: full concordance in **55.5%** (PT10 excluded for insufficient material)
- See.d® and Streck: **partial or no concordance** (4 cases), with **clinically relevant variants** (in ESR1 and PIK3CA genes) **exclusively in cfDNA**

Conclusion

In this feasibility study in MBC we show that:

- **See.d® + SmartBioSurface® workflow** is a cutting-edge platform enabling high-efficiency CTC detection and enhanced multimodal analysis
- **See.d® plasma cfDNA:** delivers mutation profiles with clinical relevance fully aligned with standard methods of analysis
- **CTC-cfDNA-tissue concordance:** highlights the power of liquid biomarkers in revealing complementary tumor insights

This **integrated analysis** propels biomarker-driven precision oncology toward broader clinical validation.

Acknowledgements

The authors would like to thank the patients enrolled in this study and their families, as well as all the staff personnel at the Division of Medical Senology, Oncology Genetics Unit and D.I.M.A. Centre at IEO (Milan, Italy). The authors are sincerely grateful to Dr. D. Cominetti for his generous and valuable contribution to this work.

References

- 1) Alix-Panabières. et al, Cancer discovery vol. 11,4 (2021): 858-873
- 2) Alavi N. et al, EJIFCC. 2020 Mar 20;31(1):21-27
- 3) Mastromarino M.G., Parini S. et al., 2023 *Biomedicines*, 11(1):153
- 4) Krol I., Schwab F.D., Carbone R., Ritter M. et al., 2021, *Br J Cancer*, 125(1):23-27
- 5) Carbone R. et al., 2006, *Biomaterials*, 27(17):3221-9 [4]
- 6) NCT06097156
- 7) Bianchini G. et al, *ESMO Open*, 2025, 10(4):104978, 407P
- 8) Bascetta L. et al. Poster, EACR 2025

CONTACT INFORMATION



contact@tethis-lab.com
Visit us at Booth 202